



## BD SurePath™ Liquid-based Pap Test Collection Procedure

**Important: See specific instructions for each device type**

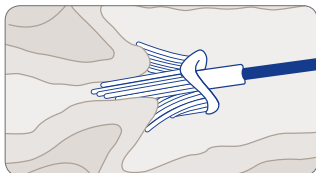

### 1. Collect

Collect the cytology sample using either a broom-like device or combination brush/plastic spatula with detachable heads. Follow manufacturer's collection instructions for detachable-head device(s).

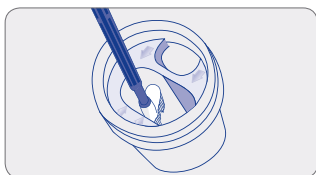









### 2. Drop

After preparing the conventional smear, use the interior rim of the BD SurePath™ Collection Vial to pull off the head of the broom-like device or snap the heads of the brush and spatula into the large opening in the collection vial.



### 3. Send

Place the cap on the vial and tighten. Send the BD SurePath™ Collection Vial to the lab for processing.

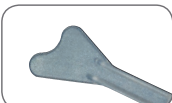Device Type	Directions For Use
<b>Rovers® Cervex-Brush®</b>	 <p>Insert the Rovers® Cervex-Brush® into the endocervical canal and rotate <b>5</b> times in a <b>clockwise</b> direction.</p> 
<b>Rovers® Cervex-Brush® Combi</b>	 <p>Insert the Rovers® Cervex-Brush® Combi into the endocervical canal and rotate <b>2</b> times in a <b>clockwise</b> direction.</p> 
<b>Cytobrush® Plus GT</b>	 <p>Insert Cytobrush® Plus GT into the endocervix until only the bottom-most bristles are exposed at the os. Slowly rotate <b>1/4 to 1/2</b> turn in <b>one direction</b>. To reduce unnecessary bleeding, do not over rotate brush.</p> 
<b>Pap-Perfect® Spatula</b>	 <p>Insert the contoured end of the Pap-Perfect® plastic spatula and rotate <b>360°</b> around the entire ectocervix.</p> 
<b>Contraindications:</b>	
<b>Rovers® Cervex-Brush®</b>	As indicated in Rovers® Cervex-Brush® product label, the broom may be used on pregnant patients up to 10 weeks gestation.
<b>Rovers® Cervex-Brush® Combi</b>	The Cervex-Brush® Combi should not be used on patients during pregnancy.
<b>Cytobrush® Plus GT</b>	Do not use the Cytobrush® Plus GT on pregnant patients or for endometrial sampling.